Eng

# Babycheck-1® WB

## Rapid self-test for detection of hCG (pregnancy) in blood

www.swisspointofcare.com  
PO Box 46, 3400 AA Bisschoven, Netherlands  
+31 85 1303 684

### GENERAL POINTS


BABYCHECK-1®WB test is a self-testing rapid test for the determination of the hCG (human Chorionic Gonadotropin) in blood sample for the diagnosis of pregnancy. While pregnancy can be detected in urine about 1 week after or missed period, blood testing can detect pregnancy as early as a few days after implantation. Pregnancy blood tests are also known to be more accurate than urine tests. This is the reason why blood/serum pregnancy tests are widely used in laboratories/hospitals. In addition, the volume of ingested liquids which may affect the urine test results (dilution of hCG) is not affecting blood test. The test accuracy is indeed very important as a pregnancy necessarily implies precautions as well as a change in lifestyle such as stopping smoking or drinking alcohol, reducing or even eliminating medications as well as certain foods. BABYCHECK-1® WB is a rapid qualitative test for the detection of the hCG in blood. In only 5 minutes, it is possible to detect an hCG concentration of 25 IU/L.

### PRESENTATION

The box contains all the material necessary to perform a test:  
• 1 sealed aluminium pouch containing: 1 test device, 1 plastic capillary pipette and 1 desiccant bag.  
• 1 sterile lancet for blood sampling.  
• 1 dropper bottle containing 1 ml of diluent.  
• 1 instruction leaflet.  
• 1 open the protective pouch when you are ready to use the test. The desiccant bag should not be used.

Not provided necessary material:

absorbent cotton and alcohol 70 % v/v or alcohol pad and liner.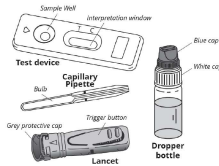


### PRECAUTIONS

- This test is exclusively intended for in vitro diagnostic use. External use **only, DO NOT SWALLOW**.
- Carefully read the instructions before performing the test. The test is only **intended for use** if the instructions are carefully respected. Follow strictly the instructions.
- Store between +4°C and +30°C. Do not freeze.
- Do not use after the expiry date printed on the label and on the protective pouch or if one of the components is damaged.
- Do not reuse the test.
- Keep out of reach of children.
- After use, all components can be discarded in a dustbin.

### TEST PROCEDURE

Prepare everything properly before starting the testing procedure. Place the content of the box on a clean, dry and flat surface (e.g. table). Then perform the test as follows:

- Wash your hands thoroughly.** Use soap and warm water. Dry your hands with clean towel.
- Prepare the test device and capillary pipette:** Take them out from the protective pouch (bear in mind) and place them in the reach of your hands (you will need them later). Discard the desiccant bag.
- Prepare the lancet.** Hold the lancet device without touching the trigger button. Unlock the cap by twisting it off. Turn until you feel it separates from the lancet device and then continue twisting it (2-3 rotations). Don't pull just twist and discard the cap when finished.
- Clean the end of the middle or ring finger** with cotton damped with alcohol. Massage the chosen finger towards the tip for 10 to 15 seconds to enhance the blood flow.
- Prick the finger:** put the lancet device firmly against the side of the previously cleaned finger, and press the release trigger button. The lancet will automatically retract into the body of the device.

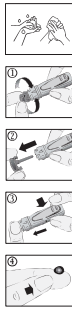Gently massage the finger to create a nice round blood droplet. ⓐ

- Collect the blood with the capillary pipette:** Gently touch the blood droplet with the open end of the capillary pipette. It must be filled until the line indicated on the capillary tube. Do not press the balloon, it fills by itself. Make another blood droplet if the line is not reached. As far as possible, avoid air bubbles. ⓑ
- Put the blood collected with the capillary pipette into the sample well of the device,** by pressing on the pipette balloon. ⓐ
- Wait 30-40 sec** for the blood being totally absorbed into the sample well.

- Add the diluent:** Unscrew the blue cap of the diluent dropper bottle (leave the white cap tightly screwed) and add the diluent as follows: Hold the diluent dropper bottle vertically and slowly add exactly 4 drops in the sample well of the device with an interval of 2-3 seconds between each drop. ⓑ

- Read the result after 5 minutes.** Do not interpret after 10 minutes.

### RESULTS INTERPRETATION

The colour and intensity of the lines do not have any importance for the interpretation of the test results.

- Positive result:** Two coloured lines appear in the window under the marks T (Test) and C (Control). The colour intensity of the T line may be clearer than the intensity of the C line. The test is positive indicating that the hCG level is than 25 IU/L or higher and that you are pregnant.

- Negative result:** Only one coloured line appears under the mark C (Control). The test is negative indicating that the hCG level is below 25 IU/L.

- Invalid result:** No line appears or a coloured line appears under the mark T (Test) without any line under the mark C. In this case, it is not possible to interpret the test, which must be considered as invalid. It is recommended to repeat the test with another BABYCHECK-1® WB test and new blood sample.

### QUESTIONS AND ANSWERS

#### How does BABYCHECK-1® WB work?

When a woman becomes pregnant, her body begins to produce a hormone, human chorionic gonadotropin (hCG), which is released in the blood where its concentration rises over time. BABYCHECK-1® WB Pregnancy test detects the hCG hormone in blood. The BABYCHECK-1® WB test uses a specific pair of antibodies detecting the hCG hormone in blood when the hormone level is, at least, 25 IU/L (determined against W.H.O.\* reference) or greater. \*World Health Organization

#### When should the test be performed?

You can use the BABYCHECK-1® WB Pregnancy test any time of day or night. BABYCHECK-1® WB test can detect pregnancy earlier than a classical urine test because the hCG hormone produced in the placenta is first released in the blood flow and then into the urine. Therefore, there is no more limitation to collect the morning first urine in order to increase the probability of an early pregnancy detection.

#### Can the results be incorrect?

The results are accurate as long as the instructions are carefully respected. Nevertheless, the result can be incorrect if BABYCHECK-1® WB test gets wet before test performing or if the quantity of blood dispensed in the sample well is not correct. The plastic pipette provided in the box makes sure the collected blood volume is correct.

Certain drugs containing hCG or which are used in combination with hCG (such as Pregnyl, Porfasi, Pergonal) and rare medical conditions may lead to a false positive result. Alcohol, analgesics, antibiotics, birth control (Serophen) or painkillers should not affect the test result.

The test may show a false positive result if you have had a miscarriage, abortion or have given birth within the past 8 weeks.

This is due to the test detecting hCG still present in your body.

### How to interpret the test (if the colour and the intensity of the T and C lines are different)?

The colour and intensity of the lines have no importance for result interpretation. The lines should only be homogeneous and complete. The test should be considered as positive whatever the color intensity of the test line (T), even weak.

### What is the line that appears under the mark C (Control) for?

When this line appears, it only means that the test has been performed well.

### If I read the result after 10 minutes, will the result be reliable?

No, the result should be read within 5 minutes after adding the diluent. The result is reliable up to 10 minutes.

### What do I have to do if the result is positive?

If the result is positive, you should consult your general practitioner and/or gynecologist to discuss about your pregnancy and next steps. Early prenatal care is important for a safe follow-up of pregnancy.

### What do I have to do if the result is negative?

If the test is negative, it is most likely that your blood does either not contain hCG or very low levels of hCG the test detects hCG concentration as low as 25 IU/L and that you are most probably not pregnant. However, you may have miscalculated the date of your period, especially if you have irregular periods. If your period does not start within a week, repeat the test. If you still get a negative result and your period has not started, you should consult your general practitioner or gynecologist. This delay could be caused by other reasons.

### What is the accuracy of BABYCHECK-1® WB?

The BABYCHECK-1® WB test is accurate. Evaluation reports show an overall agreement higher than 98% (95.2 - 99.5%) with reference methods. Although this test is reliable, false positive or false negative results could be obtained. \*\* CI 95%: 95% Confidence Interval.

### Information on pregnancy

L. Michie, L. and Cameron, S.T. Contraception (Elsevier), 2014, 89, 440.  
Web information: <https://www.health.harvard.edu/topics/women-health>  
Web information: <https://wehavekids.com/having-baby/how-Soon-Can-I-800-Test-Detect-Pregnancy> (2018).

Sterile lancet: **STERILE**

Owen Mumford Ltd  
Brook Hill, Woodstock, Oxfordshire OX20 1TU (UK)

**CE REP** GERMANY  
Owen Mumford GmbH, Alte Häge 1, 63762 Großostheim, Germany

	Read the instructions before use		For in vitro diagnostic use		Do not reuse
	Score between +4°C and +30°C		Batch number		Expiration date
	Manufacturer		Authorized EU representative		

MEDALAB, Rue de l'Expansion  
ZAT du Loudeau - Cerise - BP 181  
01006 ALENCON CEDEX (France)  
T +33 233 27 50 25 - [www.medalab.com](http://www.medalab.com)

**CHANGES DESCRIPTION**  
Changes type:

• N/A	Not Applicable (creation)
• Technical change	Addition, revision and/or removal of information related to the product.
• Administrative	Implementation of non-technical changes noticeable to the end-user.

Changes type	Changes description
N/A	Creation

Note: Minor typographical, grammar, spelling and formatting changes are not reported in the change details.

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Owen Mumford GmbH, Alte Häge 1, 63762 Großostheim, Deutschland